


Trolley - Assembly Instructions

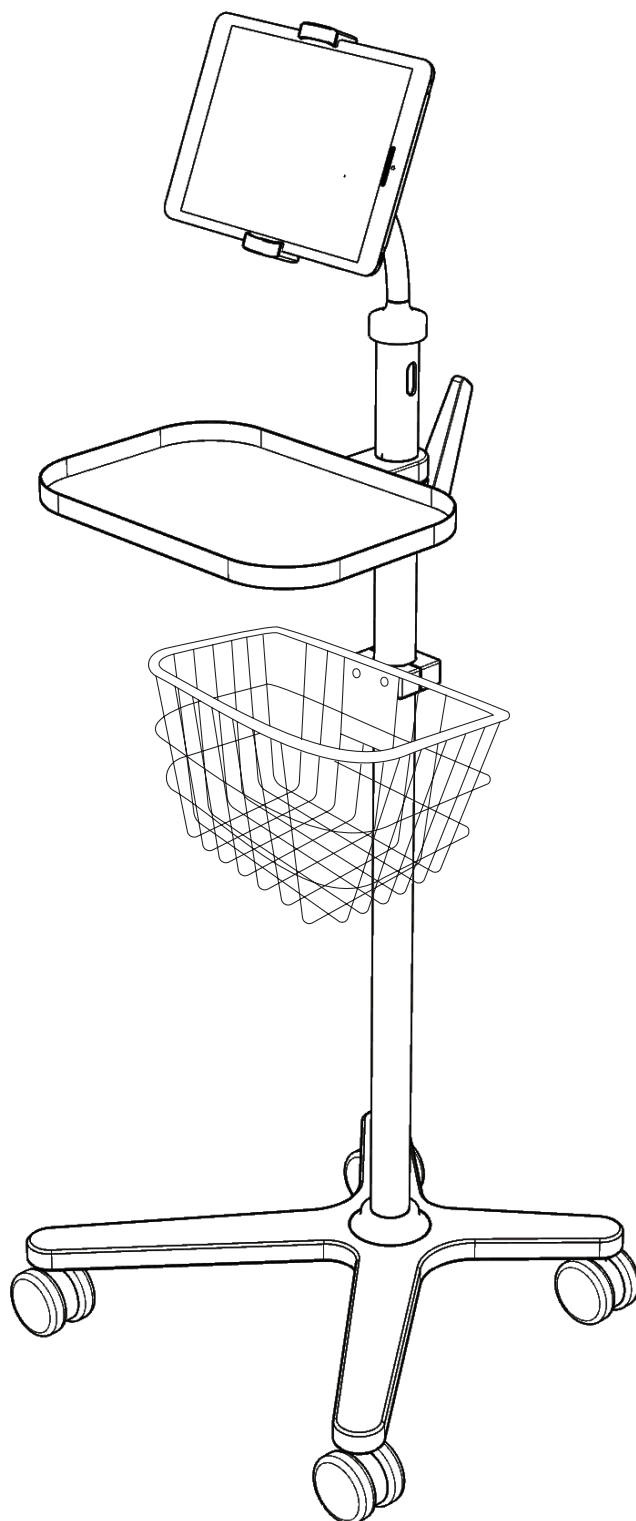


Jubo


Parts

Box 1 of 2


Tray

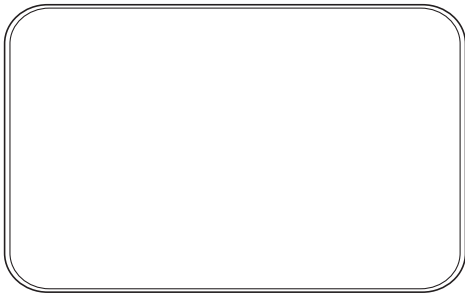


Tray Stand

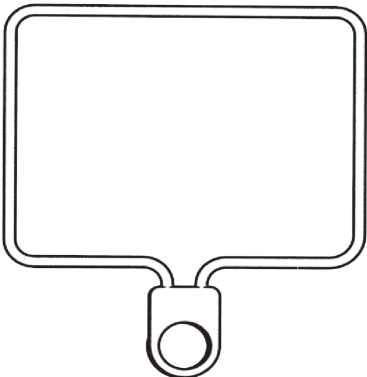

Tray Lock

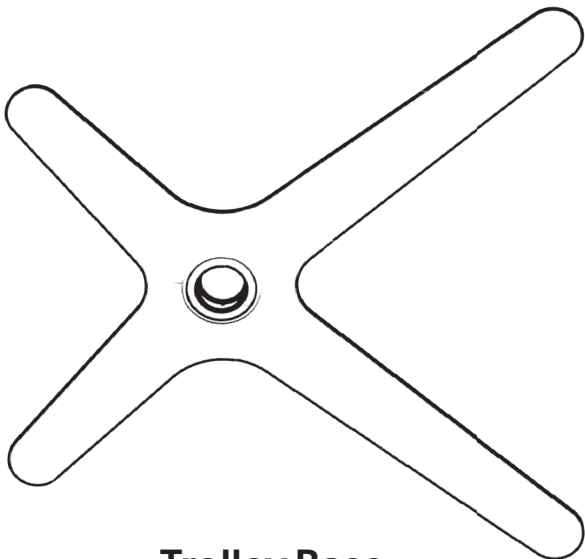


Trolley Base

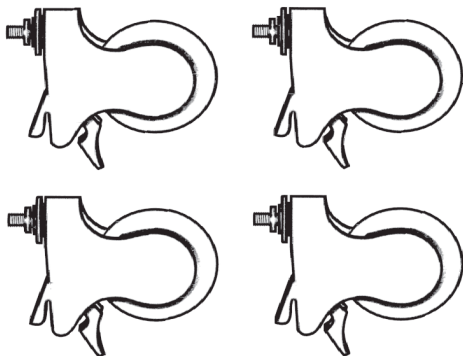


Wheels



Pole

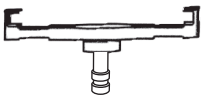

Device Holder

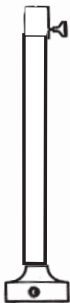


Bendable Arm


Box 2 of 2



Allen Key

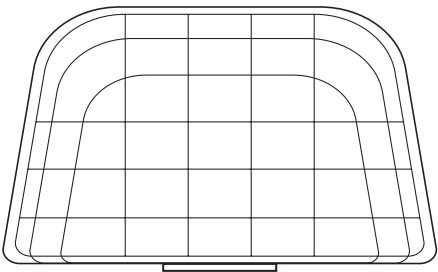

Basket

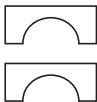


Brackets




Screws

Instructions

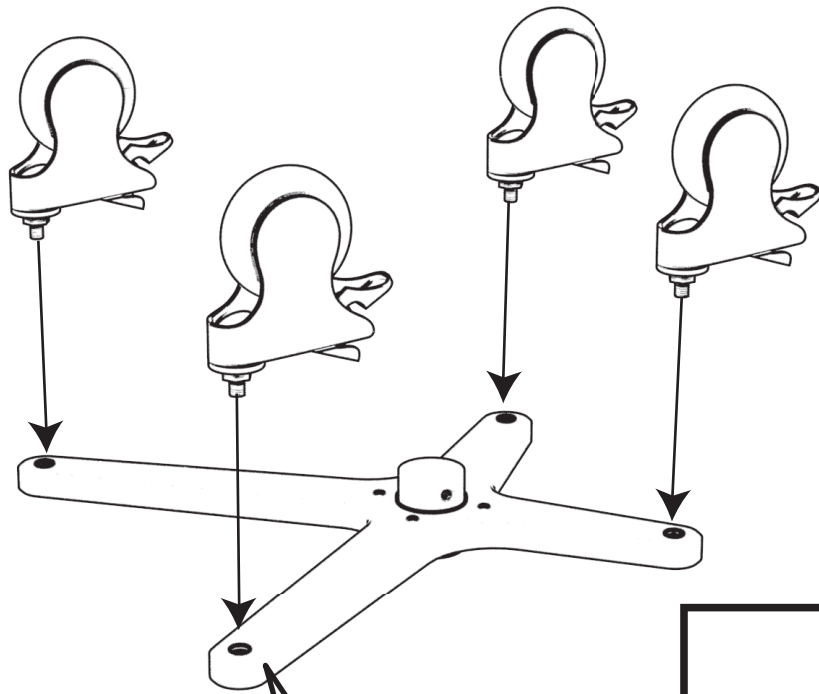


1. Place **Trolley Base** on a flat surface, with screw holes facing upward.


Lock each **Wheel**.

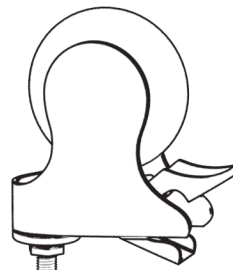

Unlocked

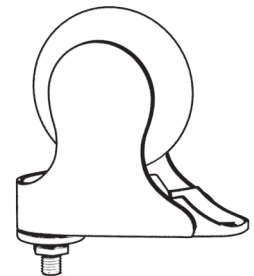


Locked

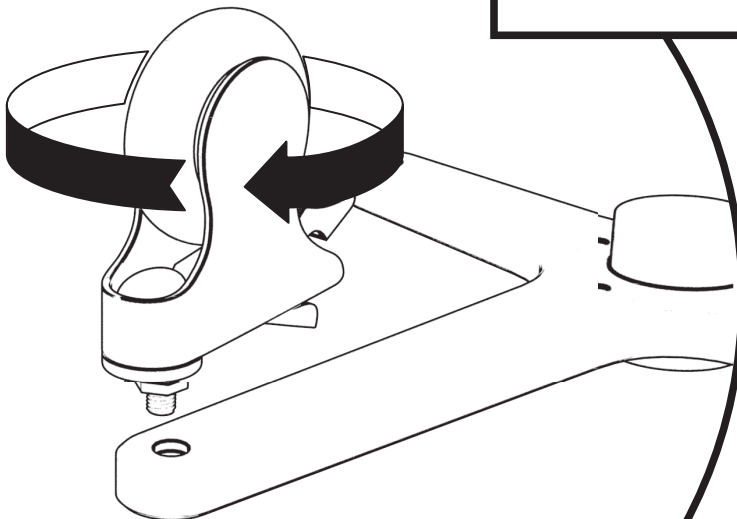


2. Screw each locked **Wheel** into the **Trolley Base** by turning them clockwise.

3. Turn **Trolley Base** right-side up.

Insert the **Pole** into the opening
on the **Trolley Base**.

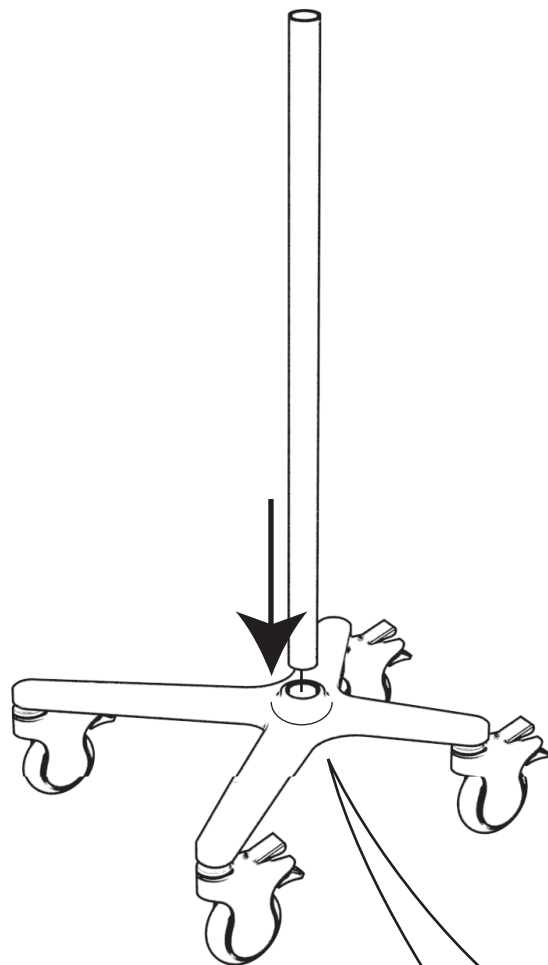


4. Use the **Allen Key** to
secure.

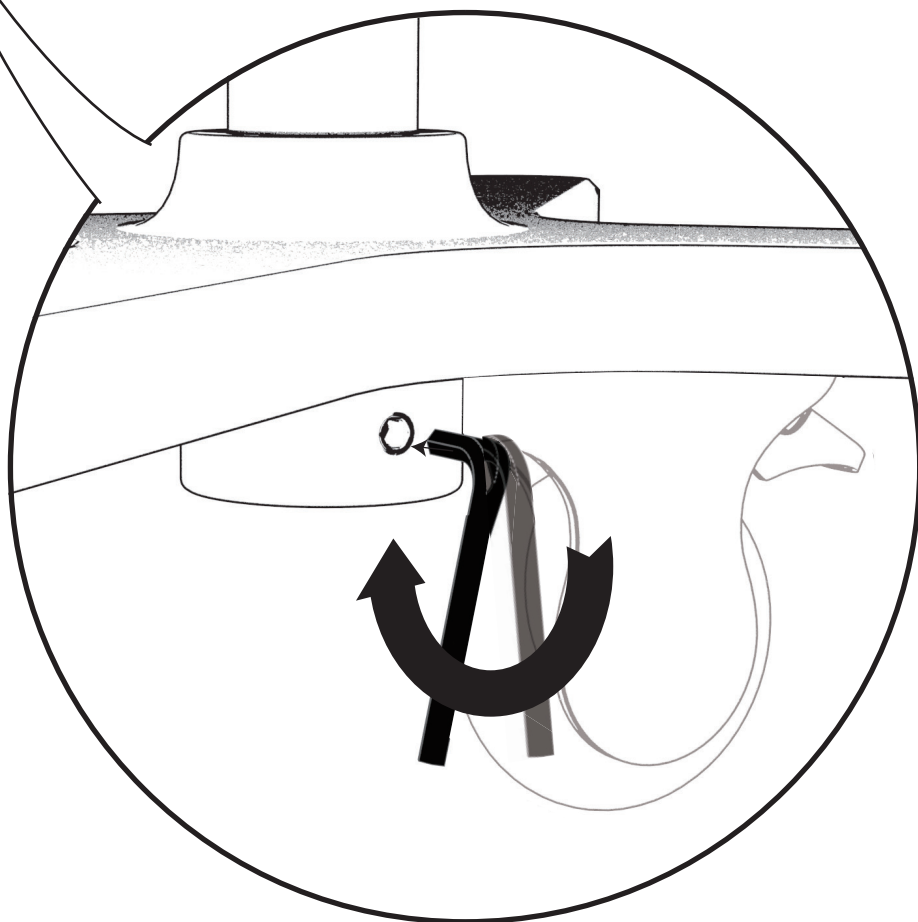

5. The **Basket** comes with a plate attached to the back of it, align the top screw holes of this plate with one **Bracket**.


Insert **Screws** through the holes from the inside of the **Basket** and secure using the **Allen Key**.

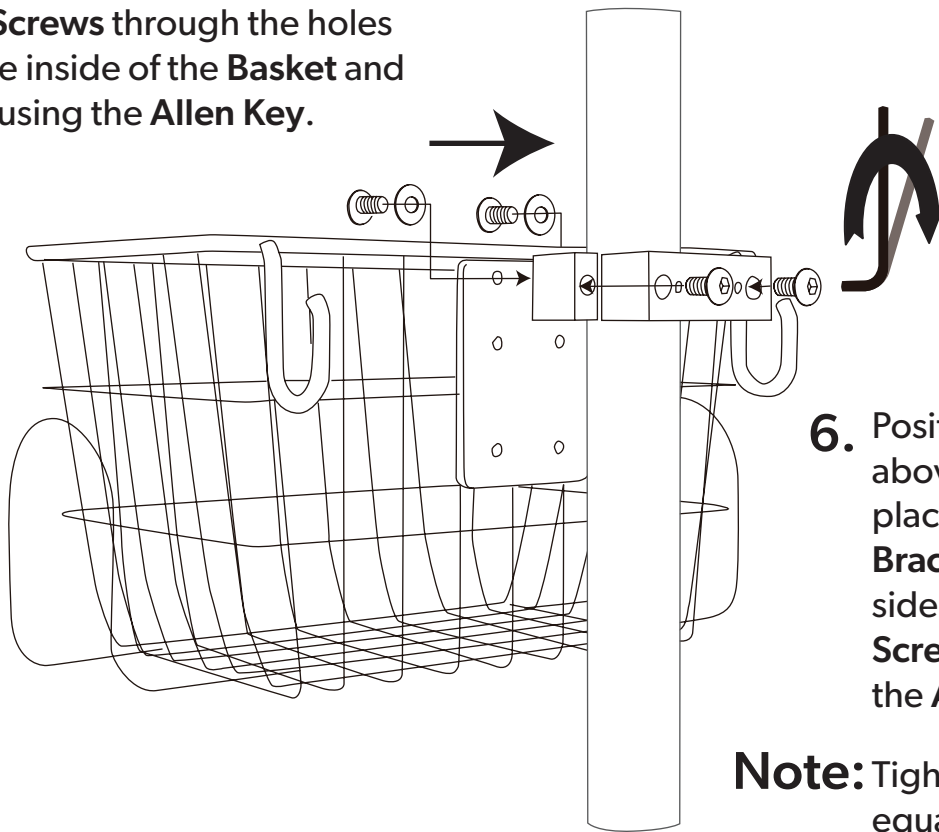

6. Position the **Basket** 12" above the **Base** and place the second **Bracket** on the other side of the **Pole**. Insert **Screws** and secure using the **Allen Key**.


Note: Tighten both screws equally to ensure **Basket** is secure enough.

7. Align the **Tray** and **Tray Stand**. Press together to secure.

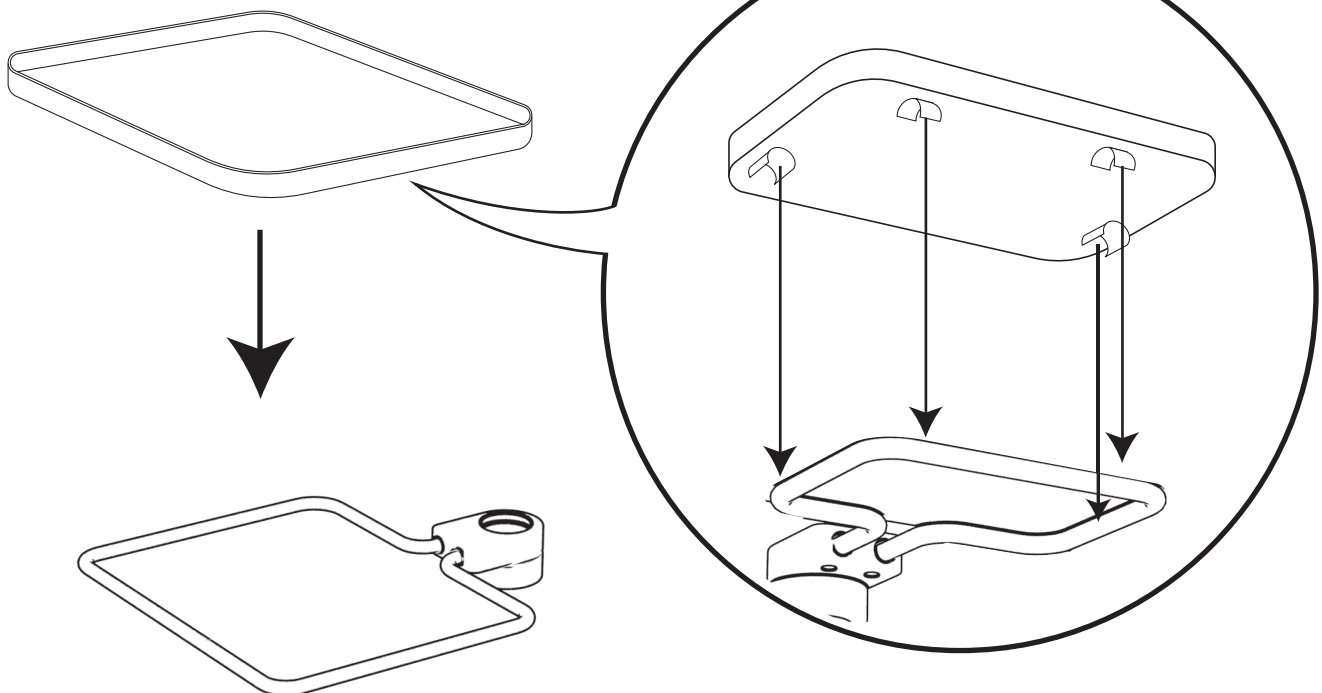


- 8.** Slide the **Tray Stand** onto the Pole.
Position 6" above the basket.


Turn **Tray Lock** counter clockwise
to secure the **Tray Stand** in place.

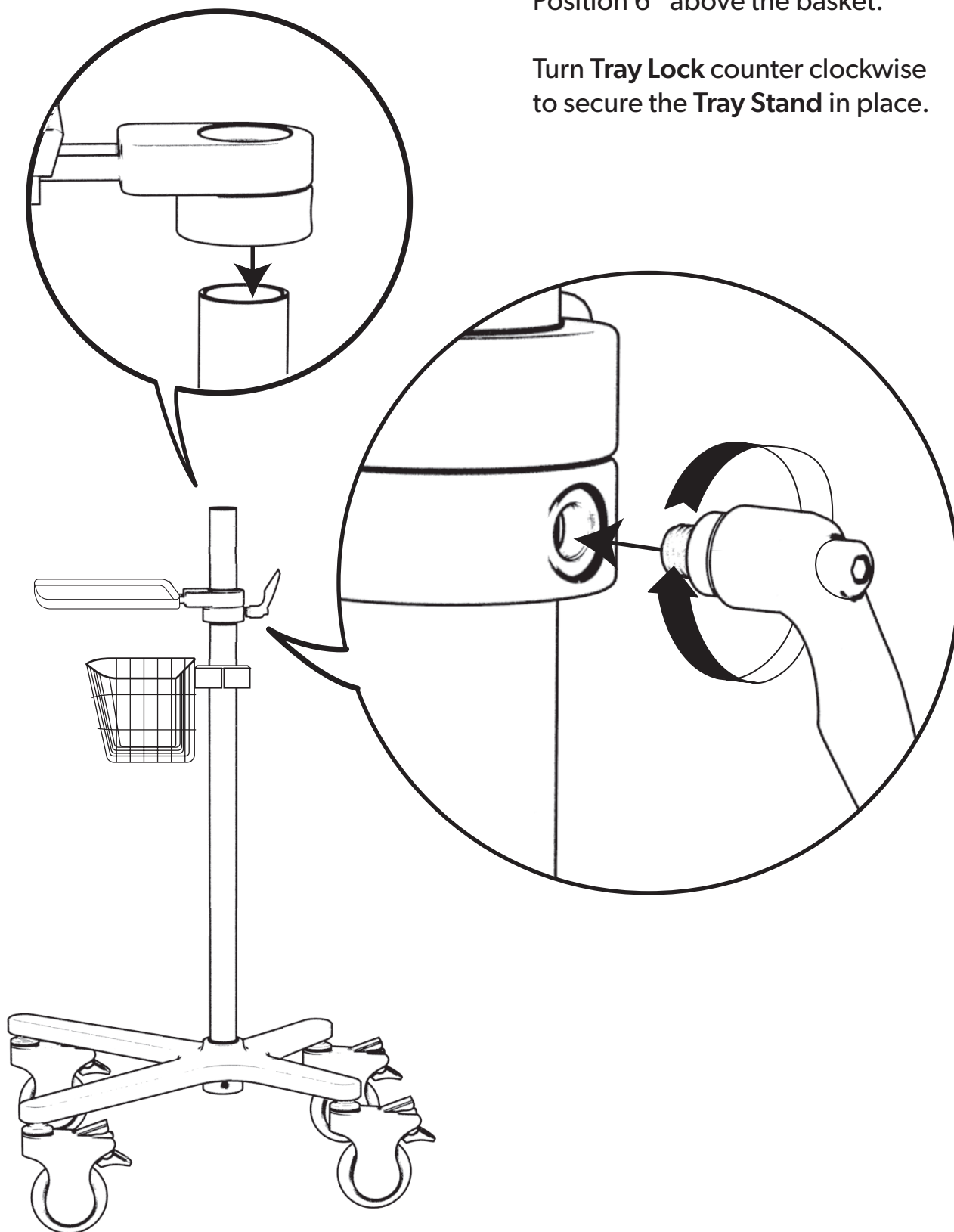

9. Place **Bendable Arm** over the **Pole** and secure using the **Allen Key**.

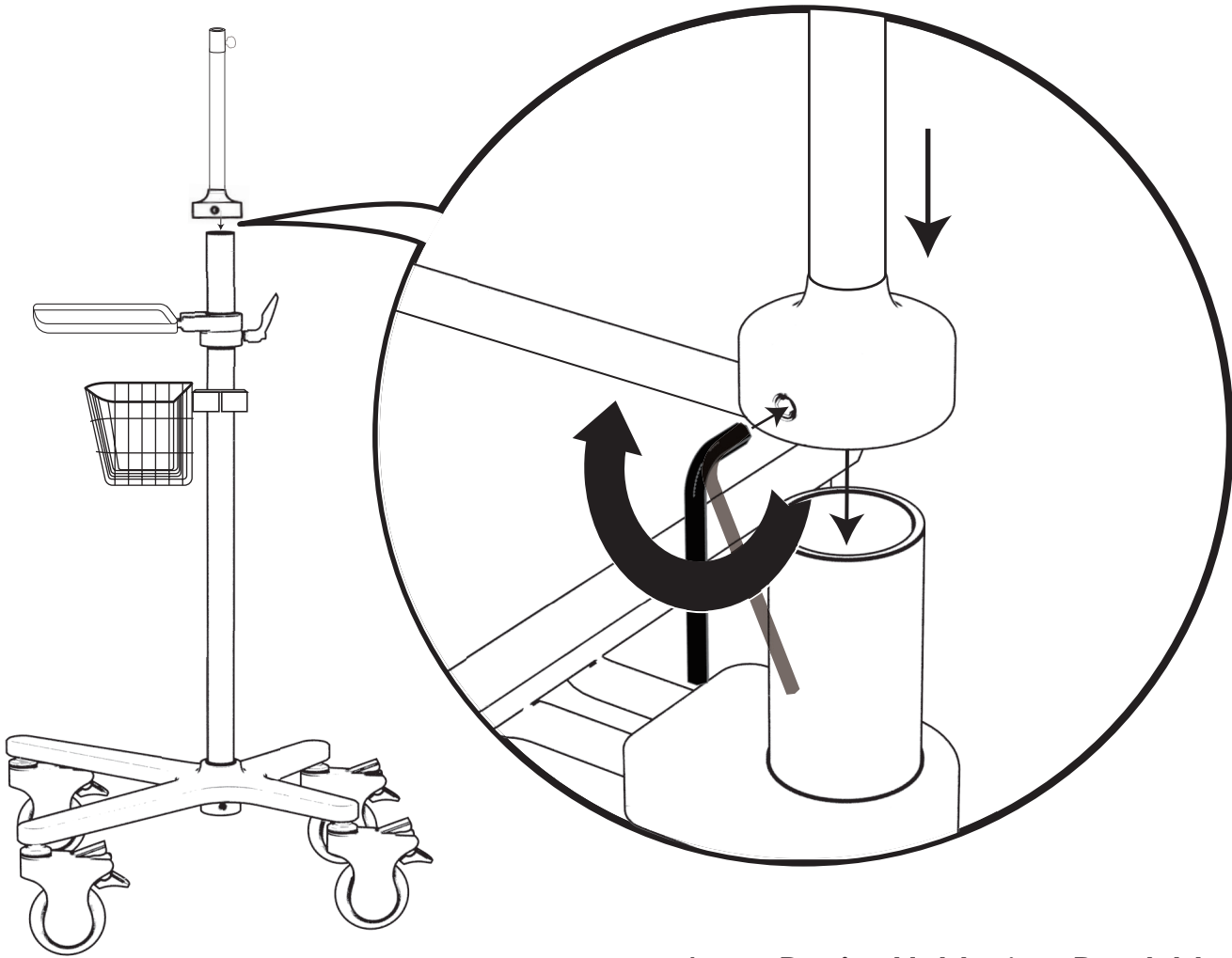


10. Insert **Device Holder** into **Bendable Arm** and secure by turning the knob clockwise. Ensure everything is secure before using to hold a device.

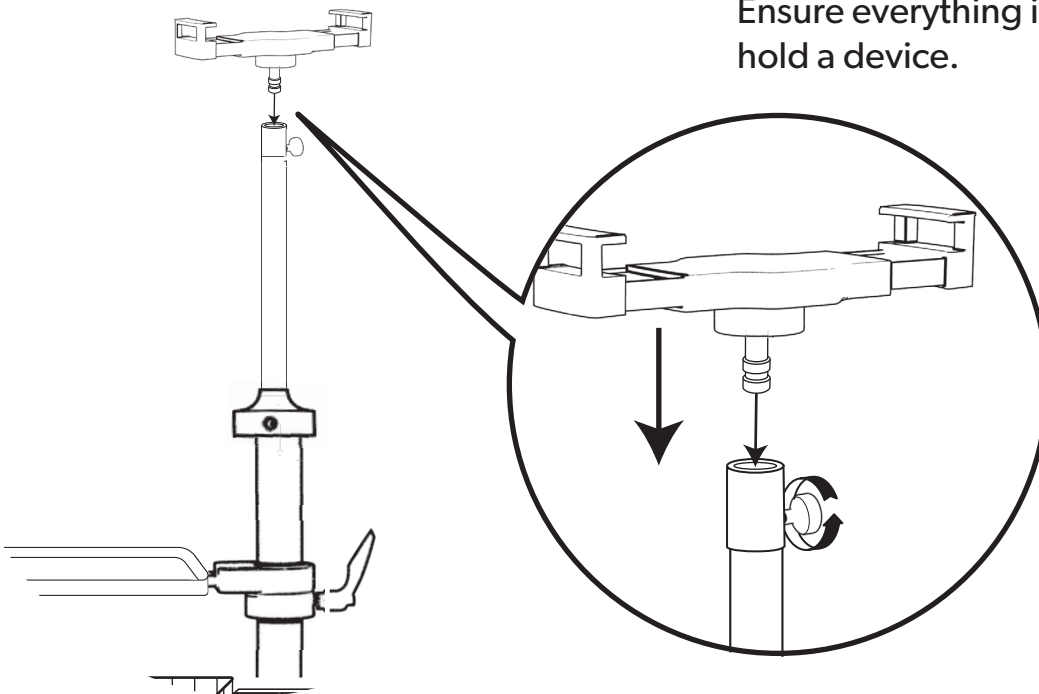


Trolley features

Device Holder - expandable to hold different device models

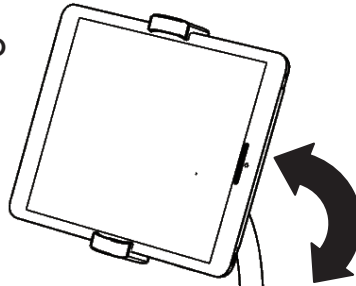


Bendable Arm - can be positioned in multiple directions

Tray Stand - height adjustable

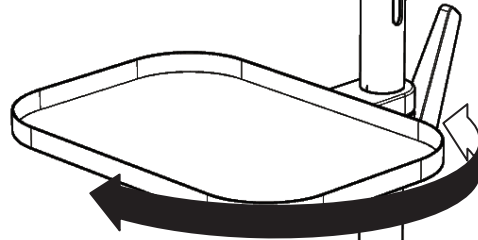


Tray - can swivel 180 degrees

Basket - height adjustable

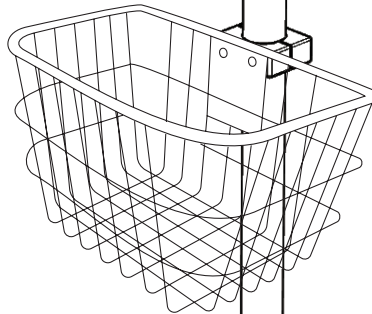

Wheels - lockable when not in motion

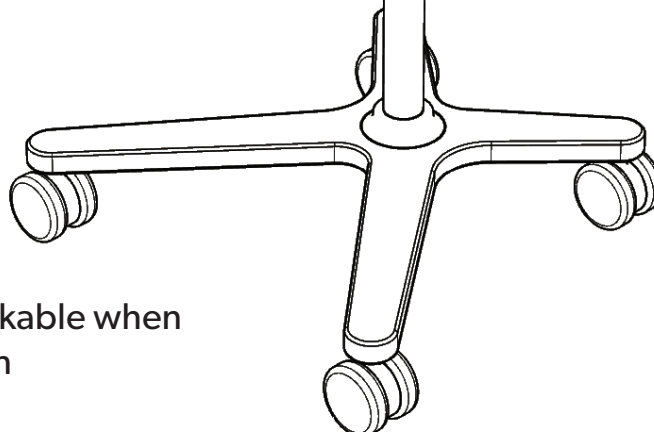

We're here for you.

Our Customer Support team offers personalized service from day one – ensuring your facilities are equipped and your team is set up for success.

1 800 906 0414 | support@jubohealth.com

For the most up-to-date copy of these instructions visit:
www.jubohealth.com/tech-specs



Connect With Us



Stay up-to-date with our blog

Jubo